

## **FloTrac System**



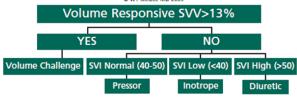

## **Stroke Volume Variation**

A sensitive indicator of preload responsiveness (on control ventilated patients)

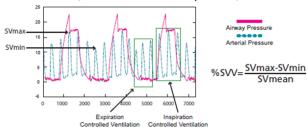

SQUARE WAVE TEST (for assessing dynamic performance): 1-Pull and release snap-tab, 2-Observe square wave on monitor, 3-Count oscillations




Optimally Damped 1.5-2 oscillations before returning to baseline.

Underdamped >2 oscillations: SBP overestimated & DBP may be low or normal Results - false widening of pulse pressure and overestimation of CO. See product insert for potential causes.

Overdamped <1 oscillation: SBP falsely low & DBP may be high or normal. Results - false narrowing of pulse pressure and underestimation of CO. See product insert for potential causes.

## **FloTrac System**

1. Cardiac Output
Blood pumped from heart in liters/min.
2. Central Venous Oxygen Saturation\*
Assessment of balance between
DO<sub>2</sub> and VO<sub>2</sub>. Lower values
indicate increased oxygen
extraction or decreased delivery. Higher levels are seen with impaired oxygen

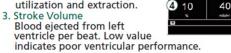

- Stroke Volume Variation (For use on control ventilated patients). Variation in arterial pulsations caused by volume changes during positive pressure inspiration. >15% may indicate hypovolemia.
- Systemic Vascular Resistance<sup>†</sup> Clinical indicator of afterload.

Vigileo Monitor Hemodynamic Parameters

Parameter	Normal Range
CO (Cardiac Output)	4.0 - 8.0 L/min
CI (Cardiac Index)	2.5 - 4.0 L/min/m²
SV (Stroke Volume)	60 - 100 mL/beat
SVI (Stroke Volume Index)	33 - 47 mL/beat/m²
SVR (Systemic Vascular Resistance)	800 - 1200 dynes - sec/cm <sup>6</sup>
SVRI (Systemic Vascular Resistance Index)	1970-2390 dynes - sec/cm <sup>6</sup> /m <sup>2</sup>
SVV (Stroke Volume Variation)	<15%
ScvO <sub>2</sub>	≥70%

<sup>\*</sup>Available when used with the PreSep catheter †Available when interfaced with CVP from appropriate bedside monitor

Rx only. See instructions for use for full prescribing information.

Edwards and Vigileo are trademarks of Edwards Lifesciences Corporation. Edwards Lifesciences, the stylized E logo, and FloTrac are trademarks of Edwards Lifesciences Corporation and are registered in the United States Patent and Trademark Office.

© 2007 Edwards Lifesciences LLC. All rights reserved. AR02456



Edwards Lifesciences LLC Irvine, CA USA · 949.250.2500 · www.edwards.com 2064