

## Further Information

### Checking Suspicious Values

If you have any doubts about the correctness of the measurement value, take a blood sample and have it analyzed using standard laboratory procedures.

### Calculating Oxygen Extraction

Oxygen extraction is the difference between the measured SpO<sub>2</sub> and SvO<sub>2</sub> values. If you are monitoring SpO<sub>2</sub> and SvO<sub>2</sub>, the monitor can calculate this value and display it as a numeric.

- To switch oxygen extraction calculation on or off, in the Setup Sp-vO<sub>2</sub> menu, select Sp-vO<sub>2</sub> and toggle between On and Off.