

MICU Arjohuntleigh Ceiling Lift Tip Sheet

- ◆ Maxi Sky 2 Single Lifts are located in MICU rooms 1, 3, 4, 5, 6, 7, 9, 10, 11, 14, 18, 21, 23, 27, and 28—these can hold up to 600 lbs.
- ◆ Maxi Sky 2 Dual Lifts are located in MICU rooms 2 and 8, and can hold up to 1000lbs.
 - ⇒ Dual lifts should be put into single mode if patient occupying room is less than 350 lbs. Dual mode should only be utilized for patients 350-1000 lbs.
 - ⇒ Instructions on how to transition lifts from single to dual and dual to single are posted in the patient belonging cabinets of MICU rooms 2 and 8.
- ◆ Repositioning slings are available in clean utility rooms (and from materials) in regular and bariatric. Repositioning slings are single patient use and are to be used to reposition the patient while in the supine position only.
- ◆ Passive Loop Slings are available in clean utility rooms (and from materials) in large and extra-large, and are also single patient use. Passive Loop Slings are to be used to get a patient out of bed to a chair in seated position. Only use slings with LOOPS when using the ceiling lifts.
- ◆ Additional lift equipment for rooms not equipped with ceiling lifts can be found in both MICU equipment rooms.