


Level 1 Fluid warmer/Rapid infuser

Controls/display

- 1 fluid warmer display panel
- 2 power and alarm test panel
- 3 reservoir level display
- 4 air detector/damp control panel
- 5 pressure chamber control panel

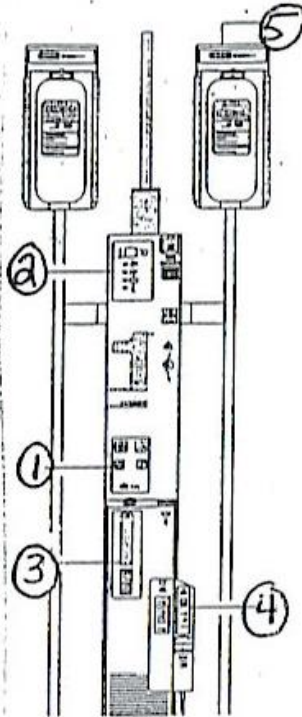

Function	Interface #	Description	Indicator/ Alarm Signal	
			Visible	Audible
Fluid Warmer Power and Alarm Test Panel	1	OFF	None	None
	2	ON	LEDs display	None
	3	Alarm Test	LEDs display	Beep
	4	Temperature Test	LEDs display	Beep
Fluid Warmer Display Panel	5	Temperature Display	LCD Readout	None
	6	Automatic Operation	Green LED	None
	7	Check Disposables	Yellow LED	Beep
	8	Add Water	Yellow LED	Beep
	9	Over Temperature	Red LED	Beep
Air Detection/Clamp Alarm and Control Panel	10	Unclamped mode/indicator	Yellow LED	Beep, short dur.
	11	Automatic Operation for Air Detection	Green LED	None
	12	Check Tubing/ Patient Line	Yellow LED	Beep
	13	Air Detected/ Clamped	Red LED	Beep
Pressurized Fluid Delivery	14	Pressure Gauge	Numbered Dial	None
	15	Pressurized	+ Symbol	None
	16	Not Pressurized	- Symbol	None