

IV Manifolds

What?

- The 6 port luer lock manifolds are now being replaced with a **3-port stopcock** manifold.
- Manifolds are used to allow compatible medication to infuse simultaneously in the same lumen of an IV.

Why?

- This is a replacement item due to supply issues.

How?

- Attach the manifold to your IV tubing and begin priming process.
- Open stopcocks to prime all ports.
- Attach a needle-free IV connector to each port.
- Add CuroS™ caps to unused connections.
- Be sure to date and initial manifolds and change per IV tubing guidelines.



Product	Supplier Info
Hi-Flo™ 3 Gang, 3-Way Manifold with MLL	Medex™: MX4313LM Smiths Medical ASD, Inc.
ONE-LINK Needle-free IV Connector	Baxter: 7N8399

