

Symbia Shut Down/Start Up Procedure

End of Day- Start the Auto Daily QC Protocol - On Thursdays, start the Weekly QC Auto QC Procedure - must remove collimators.

In the AM- Check the values for the Daily Flood and complete protocol. Proceed to the shutdown procedure.

Shut down/Start up: Options, End Session, Shutdown, Yes

- While system is shutting down, go to the PPM. Click Setup tab at bottom right corner. Press Shutdown, and Green Arrow (proceed).
- Turn power off to the UPS by holding the button in for a couple seconds. Wait 15 seconds and power back up the UPS.
- When PPM displays Ready for Shutdown, turn SNAC and CT off. Wait 15 seconds and turn SNAC back on. Push Green Go on Emergency Stop, wait for lights to show up on hand remote and then turn CT back on (off no less than 1 minute).
- Return to computer, Ctrl-Alt-Delete, wait for PPM to say Siemens and then sign on, password Esoft, click on Esoft in center of screen.

Quick Shut Down for Motion Errors

- PPM shutdown
- 3 switches off: SNAC, Motion and Camera
- CT box off
- 3 switches on: SNAC, Motion and Camera
- Emergency Stop Green button (wait until hand control lights come on)
- CT box on